

Back Cover  
14.85cm

Front Cover  
14.85cm


21cm

anko

# Wearable Breast Pump Instruction Manual

BPA  
FREE

180  
mL  
CAPACITY  
EACH CUP

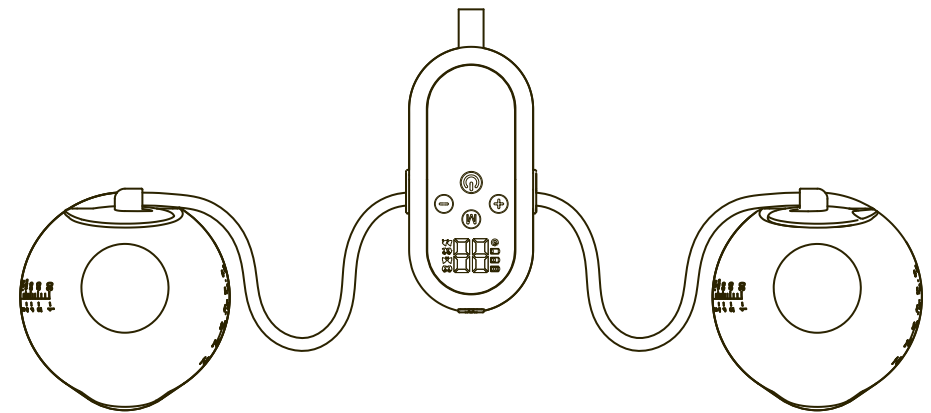

K: 43330331 | T: 68761554 (Model: JLR-78346)

MADE IN CHINA

CUSTOMER SERVICE:

KMART AU: 1800 124 125

KMART NZ: 0800 945 995

TARGET AU: 1300 753 567

The illustrations shown in this manual are for reference only. Actual product may vary slightly. The manufacturer reserves the right to change any specification or feature without prior notice.

**IMPORTANT! PLEASE READ CAREFULLY BEFORE USE  
AND KEEP FOR FUTURE REFERENCE.**

14.85cm


**WARNING:**

1. Ensure to turn off device after each use. Do not use unoriginal parts.
2. To ensure optimal personal hygiene, do not share this product with others to avoid any potential health risks.
3. The breast pump contains a non-replaceable lithium battery. In order to maintain good power storage function, please avoid long-term over-charging.
4. Do not place this item near a heat source or in direct sunlight. Damage may result.
5. Do not use this product whilst lying down or leaning back. Please stop using it when you are feeling sleepy.
6. Do not immerse the breast pump in water or any other liquid. Do not use whilst bathing. Damage may result.
7. Do not disassemble the product by yourself to avoid affecting your warranty rights.
8. In order to prolong the service life of the power cord, it is recommended you do not wind it up for storage purposes.
9. Keep all components not in use away from babies and children.
10. This product cannot be used whilst on charge. Ensure it is fully charged before use.
11. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instructions concerning use of the appliance by a person responsible for their safety.
12. Children should be supervised to ensure that they do not play with the appliance.
13. The appliance must only be supplied at safe extra low voltage corresponding to the marking on the appliance.
14. This appliance contains batteries that are non-replaceable.
15. The supplied USB charging cable should be inserted into the standard USB power source rated at 5Vd.c. 1A specification. For details of charging, refer to Power description section.

**Caution:**

1. Please read the instructions carefully before use.
2. Do not immerse this electrical device in water or a steam steriliser.
3. Ensure you do not twist the connecting tube or knot during use.
4. Please seek medical advice and guidance when using this product in less lactation stage.
5. For hygiene reasons, do not share this product with anybody else.
6. This item is not a toy. Do not allow children to use or play with it. KEEP THIS ITEM AWAY FROM BABIES AND CHILDREN.
7. Do not disassemble this product if it is damaged.
8. This appliance contains non-replaceable batteries.

14.85cm

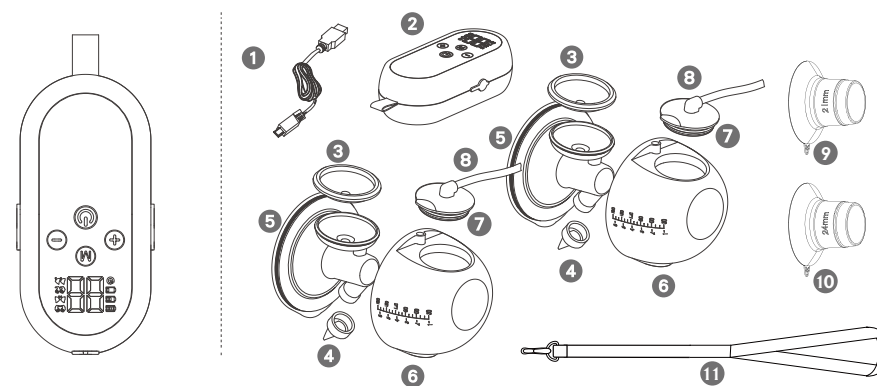**Product components:**

No.	Part Name	Material	Heat resistance temperature	Quantity
1	Type-C charging cable (Adaptor not included)			1
2	Pump			1
3	Suction Valve	Silicone		2
4	Duckbill valve	Silicone	-20°C/-4F - 160°C/320	2
5	Breast Shield	Polypropylene		2
6	Milk Storage Cups (180ml each)	Polypropylene	-20°C/-4F - 100°C/212F	2
7	Suction cap	Polypropylene		2
8	Tube	Silicone		2
9	21mm Flange	Silicone		2
10	24mm Flange	Silicone		2
11	Lanyard	Nylon		1

**Product specification:**

Power Supply	5V === 1A
Battery Capacity	1500mAh

14.85cm

14.85cm

### Product Features:

1. Hands-free portable breast pump
2. Rechargeable (USB charging cable included)
3. 4 modes each with 9 levels
  - \*single side alternating massage
  - \*single side alternating suction
  - \*double side simultaneous massage
  - \*double side simultaneous suction
4. Automatically switches from massage to suction mode
5. Anti-backflow design
6. Silicone edged flanges for comfort
7. Ultra-lightweight milk storage cups (80gm each)
8. On/off button
9. Digital display pump monitor
10. 30 minute timer function
11. Quiet: less than 62dB
12. Lanyard for easy portability.

### Assembly Method:

Note: the below illustrations are for reference only.

All accessories must be assembled well, otherwise the suction function will be affected and milk spillage will possibly occur.


Assemble the suction valve over the breast shield.



Assemble the duckbill valve under the breast shield.




Assemble the milk collecting cup according to the direction shown in the figure.

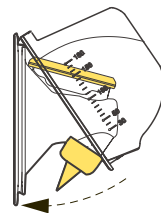


3


Press both sides firmly with both hands to ensure that it is secure.

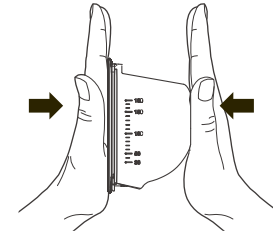

Assemble the suction cap and press it tightly to ensure that it is secure.

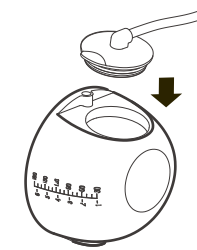


Use the provided tube to connect both sides.

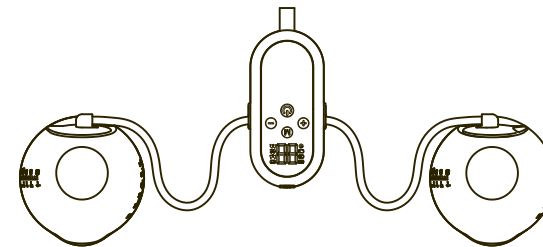

### How to wear:

Choose the correct size flange and assemble into the breast shield for each side. Place over your breast and align your nipple with the centre of the breast shield. Repeat on other side.

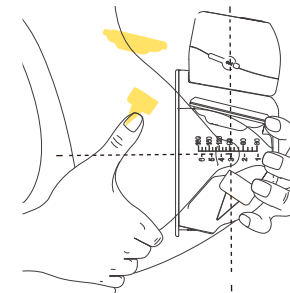


4

21cm


14.85cm

14.85cm

Put on your underwear and start pumping milk.

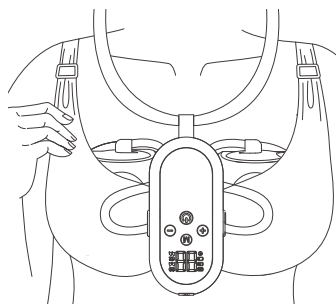

### LED display instructions:

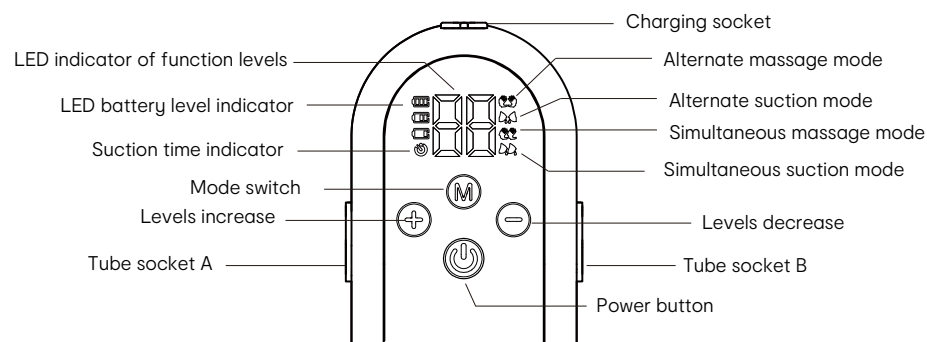

1. Before pumping, please wash your hands. It is recommended to wipe your breasts with a warm clean towel.
2. This device provides 4 modes: Alternate massage mode, Alternate suction mode, Simultaneous massage mode, Simultaneous suction mode.
3. Short press the power button to turn the device on; when the device is on, short press button to pause massage or suction, another short press button to start again. Long press button to turn off the device.  
(If the power is turned off due to insufficient power, it can be turned on only after charging)  
Levels: 1-9 for each of the 4 modes, and press (Levels increase) or (levels decrease) to adjust the suction value.
4. Short press button to switch to another mode.
5. Whenever you press power button to turn it on, the device will enter the Alternate massage mode with the level used during the last operation. User could switch the working mode by pressing button . During use, the suction value can be adjusted by (Levels increase) or (levels decrease).

5

6. This product will turn off automatically and safely after 30 minutes use.

7. The suction time indicator will be on during no operation for 9-10 seconds, the display starts to show time:

(1) Within 59 seconds, display shows in second.


(2) After 60 seconds, the display shows in minute (1-30 minutes).

### Power description:

1. The device contains 1500mAh lithium battery. One full charge takes about 4 hours.
2. When the battery capacity is lower than 30%, LED battery indicator will display a flashing light.
3. When charging, LED battery indicator displays a flashing light; When full, the full charging indicator will stop flashing.


### Tips for use:

1. The size of the breast shield will not affect the quantity of milk collected, but will affect the comfort of use.
2. The breast shield should be aligned with the centre of the breast (as shown in the below figure). Ensure that the nipple does not touch the breast shield as you may experience stimulating friction pain.
3. Please note that this item comes with 2 size flanges ( 21mm and 24mm). Please ensure you use the most comfortable fit.
4. If you feel any pain or discomfort, please turn off the power and reposition the breast shield. Restart the unit with the first level of suction. If you continue to feel pain and discomfort, please stop using this item immediately and consult your doctor.

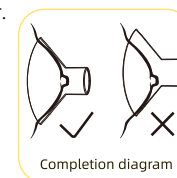

### How to pour out the breast milk:

1. After milk collection, pull out the pump first and face the breast shield upward to avoid breast milk spillage.
2. Prepare the milk storage bottle and pour out the breast milk according to the correct milk pouring method shown below.

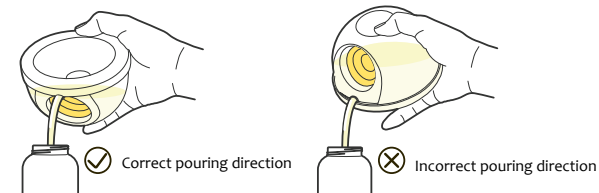

6


21cm

**How to store breast milk:**


1. Considering the expansion of breast milk after freezing, it is recommended to store 3 / 4 of the capacity of the bottle before freezing breast milk.
2. It is recommended that each milk storage capacity is about 180mL, which not only thaws faster, but also prevents waste caused by the baby's inability to consume larger quantities.
3. It is recommended to mark the date on the outside of each milk storage container.
4. It is recommended that thawed breast milk be placed in cold storage for several hours or thawed with warm water at about 40°C/ 104F. If thawed with warm water, gently shake the breast milk to prevent fat separation.
5. Thawed breast milk cannot be frozen again and must be consumed within 24 hours.

**Cleaning method:****Unit:**

1. Do not wash or disinfect in water.
2. Wipe the surface with paper towels.
3. If accidentally dropped into water or other liquids, DO NOT use this product.

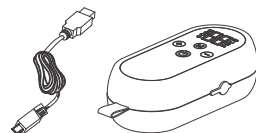**Accessories:**

1. Hand or machine washable.
2. May be sterilised by steam, ultraviolet and boiling water. Ultraviolet method may cause some yellowing, however this is acceptable. Once the hot water has boiled, please turn off the heat source and wait for 3 minutes before use.
3. Allow parts to air dry.
4. Silicone accessories are recommended to be soaked in hot water and cleaned regularly.

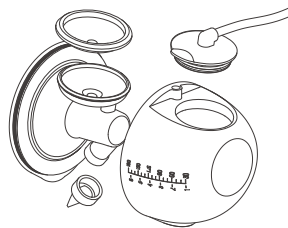**Trouble shooting Methods:**

If the appliance is not working, check the below:

1. Check the battery power; replace the power adaptor; make sure that the adaptor interface is inserted into the host's DC interface correctly.

If the product has insufficient suction, check the below:

1. Replace the parts of the closed valve; check whether the closed valve is installed correctly or not; usually put the closed valve in boiling water for 3 minutes and assemble it, to help restore suction.

2. Ensure that the breast shield is securely rested on the breast and that there is no air leakage.
3. If the room temperature is below 15°C, allow the unit to continue to run for a further 30 seconds as this may provide time for the breast suction operation.

**12 Month Warranty**

Thank you for your purchase.

Your new product is warranted to be free from defects in materials and workmanship for the period stated above, from the date of purchase, provided that the product is used in accordance with accompanying recommendations or instructions where provided. This warranty is in addition to your rights under the Australian Consumer Law.

For New Zealand customers, this warranty is in addition to statutory rights observed under New Zealand legislation.

We will provide you with your choice of a refund, repair (where possible) or exchange (availability dependent) for this product if it becomes defective within the warranty period. The business will bear the reasonable expense of claiming the warranty.

This warranty will no longer apply where the defect is a result of alteration, accident, misuse, abuse or neglect.

Please retain your receipt as proof of purchase and contact our Customer Service Centre as listed below, for the entity from which you purchased this item, for any difficulties with your product.

Warranty claims and claims for expenses incurred in returning this product can be addressed to the respective Customer Service Centre.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

**Contact for Kmart purchases**

Kmart Australia Ltd.  
C/- Customer Service Centre  
690 Springvale Road,  
Mulgrave Vic 3170  
Customer Service: 1800 124 125  
(Australia) or 0800 945 995  
(New Zealand) or via  
Customer Help at kmart.com.au

**Contact for Target purchases**

Target Australia Pty Ltd  
C/- Customer Service Centre  
2 Kendall Street, Williams Landing,  
Vic, 3027  
Customer Service: 1300 753 567 or  
via Customer Help at  
target.com.au/help/contact-us