


# Assembly Instructions

## Components

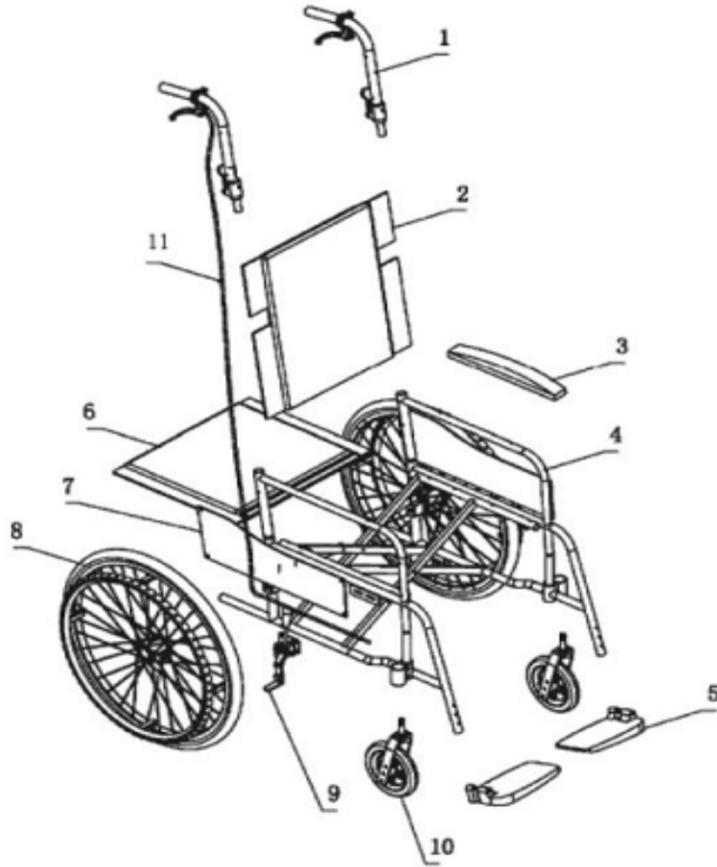

- 1, Handle tubing
- 2, Backrest
- 3, Armrest
- 4, Main frame
- 5, Footrest
- 6, Seat

- 7, Apron board
- 8, Rear wheel assembly
- 9, Brake assembly
- 10, Front wheel assembly
- 11, Brake wire

## Safety tips

Please read carefully before using your wheelchair.

1. Before entering or leaving the chair, engage the dual wheel locks against the tyres on both rear wheels. Make sure the front castors are in the forward position before transferring into or out of the chair.
2. Do not move forward on the seat while leaning forward out of the chair.
3. Do not step on the foot plates when moving, as this could cause tipping.



Max Weight Capacity: 110 kg

# Assembly Instructions



**Step 1:** How the wheel chair looks when taken from the carton.



**Step 2:** To open the chair, face the front of the chair, place hands on seat rails and push down and out, the seat will open.



**Step 3:** Lift back support to upright position. The lock will engage automatically.



**Step 4:** Turn down the left and right footrests.

# Assembly Instructions



**Step 5:** Take the belt from the back bag of the chair.



**Step 6:** Stick the belt to the front of the chair. Now the wheel chair is ready to use.



**Step 7:** When you don't want to use the chair, you can fold down and store, pull up on locks simultaneously and carefully fold down the back support.



**Step 8:** The wheel chair will need maintenance when using a period of time to ensure safety.